

**Attention**

1. Please take off shoes before taking measurement to avoid scraping or scratching the glass platform.
2. The glass platform may be pretty slippery when wet, thus please make sure both the glass and your feet are dry before using.
3. Do not immerse the scale in water or use chemical/abrasive cleaning agents to clean it.
4. Do not strike, shock drop or disassemble the scale.
5. Suitable for family use only.
6. Please check the battery if the scale malfunctions. Charge it if necessary.

For any query contact:- [dr@drtrustusa.com](mailto:dr@drtrustusa.com)

-3-

**Eco Recharge Zeus Scale****USER'S MANUAL**

Thank you for Purchasing Dr. Trust Zen Recharge scale. please read all instructions carefully before use.

**Key features:**

- High precision strain gauge sensors system.
- Tempered-glass platform.
- Capacity: 6 kg-150 kg
- Division: division is 50g if weight is <100kg, division is 0.1 kg if weight is 100kg~150kg
- LED display
- Auto on.
- Auto zero & Auto off.
- Low battery and over load indication.
- Free switch within different units:kg/st/lb.
- Battery function: USB charge battery.




-1-

**Operation**


- Select kg/st/lb by pressing unit button at the top of LED
- Place Scale on a hard, flat surface.
- Step on the platform & remain still until weight appears.
- The scale will automatically turns off.

**Caution**

- To ensure accuracy, we suggest you take measurement at a fixed time every day (e.g.7:00pm-9:00pm).
- When LED indicates "LO" charge the scale with USB Cable connecting it with Android Connector.

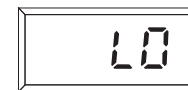

- When the Scale is overloaded, "Err" will appear on the screen

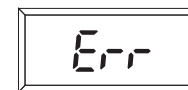

-2-