ABOUT US

Driven by the passion for innovation, we at Dr Trust endeavour to provide our customers with the latest medical inventions with an objective to promote good health and wellness all around the world. All the medical devices and health monitors provided by Dr Trust are supported by accurate, latest and ground breaking technologies, innovated at our headquarter in NY, USA. All our products adhere to the most stringent CE and FDA guidelines and are strongly recommended by doctors and health practitioners. Our products are designed in the utmost exemplary ways to ensure that their accuracy and convenience are unrivalled. The ease of their use and operation makes them even more suitable for users of all age groups.

Dr Trust strives to enhance the quality of lifestyle by providing with the most trusted and innovative health care and wellness products. Being a renowned global leader in health care products, Dr Trust ensures that our technically efficient team works dynamically and tirelessly to provide the best of the medical devices to our clients. The products that we have to offer are suitably designed for use at homes, laboratories and hospitals.

Our ground breaking solutions allow you to monitor your health in the easiest ways possible. In today's era when all of our lives are too hassled to handle, it becomes a bit difficult to pay attention to our health. But it has now become easier with the coming of the monitoring devices which can be conveniently used at homes and even on the go.

We bring to you a variety of best self medical devices, trusted and used by Doctors, medical professionals and home users all over the world.

Dr Trust

Elegance Personal Scale - 514

QUICK START GUIDE

Step 1

Install the battery and place the scale on a hard and a flat surface

Step 2

Set your desired measurement unit and initialize the scale for autocalibration before first-time use

Step 3

Step on the platform and stand still to get accurate weight measurement.

Step 4

The weight would be displayed in seconds.

Step 5

When you step off the scale, it will turn off automatically.

OVERVIEW

The Dr Trust Elegance Personal Scale is a perfect health partner for those who want to achieve weight management goals. It is a high precision strain gauge sensor technology that allows you to track your weight accurately. The scale is auto-calibrated and does not require added effort to activate. Its glass platform is wide enough to accommodate large sizes feet comfortably while four anti-skid feet provide enough hold to the scale to stay on the ground.



OPERATION

Step 1. Power Supply

Open the battery cover and install 2x1.5V AAA battery

Step 2. Place the Scale on a Hard, Flat Surface.

Place the scale on optimal flooring surfaces include ceramic tiles, concrete, or hardwood as uneven floors like carpeted may affect the readings.

Step 3. Choose Your Desired Measurement Unit

You can choose the measurement unit among kilogram and pound by clicking the unit button of your scale.

Step 4. Initialize the Scale.

If you are placing the scale down for the first time or moving your unit you must auto-calibrate the scale.

Step 5. Step on

Place both feet on the center of the platform and standstill

Step 6. Get Weight Readings

Your weight will appear on the LCD.

Step 7. Step off Your Scale

It will reset to "0.0" and turn off automatically.

AUTOMATIC TURN OFF

The scale will turn off automatically after 10 seconds of non-use.

Note

Please keep in mind the first reading may not be accurate so it can be disregarded.

ERROR CODES (Overload and Low Power Indication)

- ${\bf 1.LO-When}\ the\ LCD\ indicates\ "Lo",\ please\ open\ the\ battery\ cover\ and\ replace\ batteries\ with\ correct\ polarity\ and\ negative.$
- 2.OL-- Maximum capacity exceeded. When the scale is overloaded (over 180kg/396lbs) "OL" will appear on the screen.

■ PRODUCT SPECIFICATION ●

- 1. High Precision Strain Gauge Sensor
- 2. Max Capacity:180kg/396lbs
- 3. Division:0.1kg/0.2lb
- 4. Unit: kg/lb/st:lb
- 5. Auto on & Auto-off
- 6. Low Power & Overload Indications

CARE & MAINTENANCE

- Store and operate the scale in a safe and dry location-
- $\bullet \qquad \text{Do not expose it to direct sunlight, extreme hot/cold or humidity.}$



- Always clean it with a soft, damp cloth. Avoid abrasive cleaners and never sink or submerge in water.
- Always remove batteries if not in use for three months or longer.

OPERATION

- Please place the scale on a hard & flat surface when in use to ensure the greatest accuracy and service life.
- Clean the scale with a wet cloth, but do not allow water to seep into the unit and body scale.
- Please remove the battery from the scale if it has not been used for a long period.
 Do not stack anything on the scale during non-use.
- Never use abrasive cleansers or chemical solvents, as they may mar the casing.
- Do not put the scale in an upright position, only appropriate flat.
 Do not drop or beat the scale, this may damage the sensor and glass.
- Do not drop or beat the scale, this may damage the sensor and glass.

 To ensure the life of the scale, please do not put in a wet or hot environment.
- If the scale does not work, please check the battery, and ensure the battery is with the correct polarity.
- Please check if the battery needs to be changed when the scale has been used regularly replace the battery, and try to start again.
- This scale is for family use only and should not be used commercially for any reason at all.
- Please do not open the rear cover if the user is not professional.
- To switch weight unit exchange display between kilograms (kg) and pounds (lb), please press the "Set/Unit" button located on the bottom of the scale.

CUSTOMER SUPPORT •

CONTACT ADDRESS

USA

Nureca INC.USA

276 5th Avenue, Suite 704-397, New York (NY) - 10001, USA INDIA

Corporate Office (Mumbai)

Nureca Limited

128 Gala Number Udyog Bhavan, 1st Floor Sonawala Lane, Goregaon East

Mumbai City Maharashtra 400063 Contact us India: +91-7527013265 /+91-9356658436

Website: www.drtrust.in

Corp Website: www.nureca.com
Email: customercare@nureca.com

Connect with us on social networks
Facebook: @drtrust

Instagram: @drtrustisin Youtube: NurecaUsa

COPYRIGHT[©]2021 NURECA LTD ALL RIGHTS RESERVED



Scan to View
Product Demo Video
www.drtrustusa.com/514